

Setup Guide

1 Unpack



A400E DECT 6.0
Amplified Cordless
Expandable Handset

Please see the *ClearSounds A400 DECT 6.0 Amplified Cordless Phone User's Guide* for additional instructions.



Telephone handset



A400E charger



AC power adapter assembly



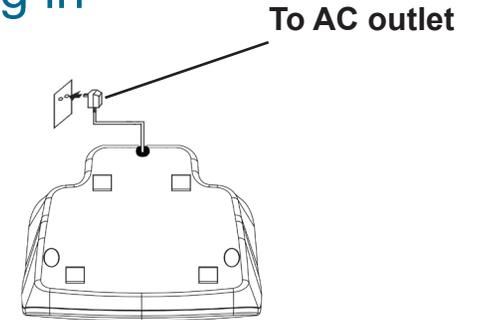
Three rechargeable AAA
NiMH batteries



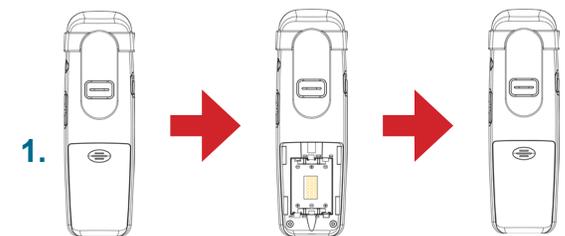
Carrying clip

2 Power

Plug in



Charge



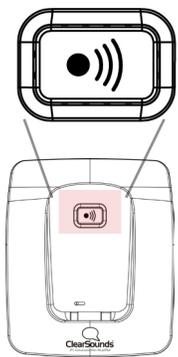
1.



2.

Prior to use, charge for 24 hours.

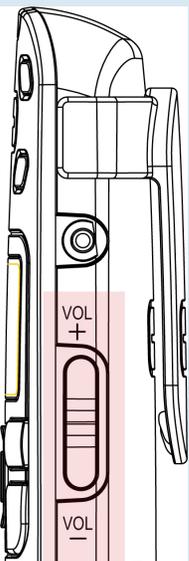
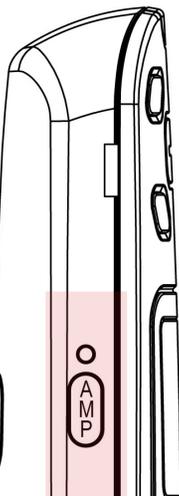
3 Register

	<p>1. On the unregistered A400E handset, press and hold  beneath Menu on the screen, until Press and hold base Page key appears, and then release the handset key.</p>
	<p>2. On the A400 base to which you are registering, press and hold  for 10 seconds, and then release the key.</p> <p>When the handset is registered to the base, the handset number appears on the handset screen.</p>

4 Call

Place	Answer	End
<p>1. Dial .</p> <p>2. Press .</p>	<p>1.  Lift.</p> <p>2. Press .</p>	<p>1. Press .</p>

5 Volume

Volume	Amplify	Equalize
 <ul style="list-style-type: none"> • Press + or -. 	 <ul style="list-style-type: none"> • Press to turn on or off. 	<p>1. Press .</p> <p>2. Press  to select Treble, Flat or Bass.</p> <p>As you select, a graph appears on the screen.</p>

ClearSounds 1-800-965-9043